

- Perform inspection of N95 mask including seal test.
- For new masks, write your first initial and last name on the right side of the front of the mask with a Sharpie.
- Write number on front left side of mask indicating 1st or 2nd use.
- Write your name on a paper bag and use for mask storage during the day.

In Work Area Prior to Leaving:

- Obtain a Tyvek sterilization pouch on the unit where you are working.
- Perform hand hygiene and don gloves.
- LEGIBLY write LAST NAME, first name and date on pouch flap with Sharpie.
- Place mask in sterilization pouch, seal tightly (to assure sterilization) and throw away brown paper bag.
- Doff gloves and perform hand hygiene.

Proceed to Departmental Dropoff:

- Bins for mask sterilization drop off will be located in your work area.
- Place your marked, sealed pouch in the bin.
- Perform hand hygiene.